



Aperio GT 450 Release 1.3.1 Patch Notice

Leica Biosystems is always looking for ways to further improve our customers' experience using our products. Our software patches often include new features, but also include improvements based on customer input and our own internal testing. Please make sure to read below for the details of this patch so you can get the best out of your product. Thank you for being a customer of Leica Biosystems.

New Features in this Release

Case #	Description
28508	Aperio GT 450 and Aperio GT 450 SAM Software now supports 2x pyramid level downscaling of scanned images. This enables third-party viewing software to display all available magnification levels with less interpolation of the image data.

Improvement Included in This Patch

**Indicates changes that were previously released as customer hot fixes.*

Case #	Description
28473	A compatibility issue with Aperio GT 450 SAM version 1.3 and the Aperio GT450 scanner versions prior to version 1.3.0 caused barcode regular expressions to be non-functional. This issue was resolved by adding fields to allow for backward compatibility.
28457*	There was an issue that caused scans to abort intermittently. This issue was resolved by reverting the initialization sequence for the scanner's framegrabber to the way it was implemented prior to version 1.3.0.
28469	The Aperio GT 450 Autoloader firmware version displayed as "0.0.0.0" on version 1.2 scanners. This issue was resolved, and the firmware version now displays correctly.

Aperio GT 450 Release 1.3.1 Patch Notice
Part Number/Revision: PN-0050, Revision A | Date: February 2024

This document applies to Aperio GT 450 and Aperio GT 450 Scanner Administration Manager version 1.3.1 and later.


Copyright Notice

- ▶ Copyright © 2024 Leica Biosystems Imaging, Inc. All Rights Reserved. LEICA and the Leica logo are registered trademarks of Leica Microsystems IR GmbH. Aperio, GT, and GT 450 are trademarks of Leica Biosystems Imaging in the USA and optionally in other countries. Other logos, product and/or company names might be trademarks of their respective owners.
- ▶ This product is protected by registered patents. For a list of patents, contact Leica Biosystems.

User Resources

- ▶ For the latest information on Leica Biosystems Aperio products and services, please visit www.LeicaBiosystems.com/Aperio.

Contact Information – Leica Biosystems Imaging, Inc.

Headquarters	Customer Support
 Leica Biosystems Imaging, Inc. 1360 Park Center Drive Vista, CA 92081 USA Tel: +1 (866)-478-4111 (toll free) Direct International Tel: +1 (760) 539-1100	Contact your local support representative with any query and service request. www.leicabiosystems.com/contact-us/

Revision Record

Rev	Issued	Sections Affected	Detail
A	February 2024	All	New document.

For research use only. Not for use in diagnostic procedures.



23GT450
23GT450SAM
23GT450SAMSW
23GT450ZSTACKSW



815477020228
815477020464
815477020471
815477020563