



Mirth Connect 4.4.2 Upgrade for Aperio SAM DX Server and Aperio GT 450 DX on Windows Server 2016 / 2019 Patch Notice

Leica Biosystems is always looking for ways to further improve our customers' experience using our products. Our software patches often include new features, but also include improvements based on customer input and our own internal testing. Please make sure to read below for the details of this patch so you can get the best out of your product. Thank you for being a customer of Leica Biosystems.

Mirth Connect 4.4.2 upgrade for Aperio SAM DX Server and Aperio GT 450 DX on Windows Server 2016 and Windows Server 2019

Aperio SAM DX Server and Aperio GT 450 DX now support the upgradation of Mirth Connect 4.4.2 on Windows Server 2016 and Windows Server 2019, and no longer support Mirth versions 4.4.1 or earlier.

Mirth Connect and Administrator Launcher

Mirth Connect software was tested on Windows Server 2016 and Windows Server 2019 with no issues for operation.

Mirth Connect:

<https://s3.amazonaws.com/downloads.mirthcorp.com/connect/4.4.2.b326/mirthconnect-4.4.2.b326-windows-x64.exe>

Administrator Launcher:

<https://s3.amazonaws.com/downloads.mirthcorp.com/connect-client-launcher/mirth-administrator-launcher-latest-windows-x64.exe>

This patch notice serves as evidence that Leica Biosystems conducted successful testing to adequately ensure security of Aperio SAM DX and Aperio GT 450 DX for release as required by applicable Leica Biosystems Imaging Quality Management System (QMS) that is certified to MDSAP/ISO 13485 and GMP, and consistent with current cybersecurity best practices.

Please direct any concerns or questions to Leica Biosystems Technical Services at TechServices@leicabiosystems.com.

Mirth Connect 4.4.2 Upgrade for Aperio SAM DX Server and Aperio GT 450 DX on Windows Server 2016 / 2019
Patch Notice

Part Number/Revision: PN-0051, Revision A | Date: January 2024

This document applies to Aperio GT 450 DX and Aperio SAM DX version 1.1 and later.


Copyright Notice

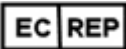
- ▶ Copyright © 2024 Leica Biosystems Imaging, Inc. All Rights Reserved. LEICA and the Leica logo are registered trademarks of Leica Microsystems IR GmbH. Aperio, GT, and GT 450 are trademarks of Leica Biosystems Imaging in the USA and optionally in other countries. Other logos, product and/or company names might be trademarks of their respective owners.
- ▶ This product is protected by registered patents. For a list of patents, contact Leica Biosystems.


User Resources

- ▶ For the latest information on Leica Biosystems Aperio products and services, please visit www.LeicaBiosystems.com/Aperio.

Contact Information – Leica Biosystems Imaging, Inc.

Headquarters	Customer Support
 <p>Leica Biosystems Imaging, Inc. 1360 Park Center Drive Vista, CA 92081 USA Tel: +1 (866)-478-4111 (toll free) Direct International Tel: +1 (760) 539-1100</p>	<p>Contact your local support representative with any query and service request.</p> <p>www.leicabiosystems.com/contact-us/</p>

European Union Authorized Representative	United Kingdom Responsible Person
 <p>CEpartner4U Esdoornlaan 13 3951 DB Maarn The Netherlands</p>	<p>Leica Microsystems (UK) Limited Larch House, Woodlands Business Park Milton Keynes, England, United Kingdom, MK14 6FG</p>

Importers	
 <p>Leica Biosystems Deutschland GmbH Heidelberger Straße 17-19 69226 Nussloch, Germany</p>	<p>Leica Microsystems (UK) Limited Larch House, Woodlands Business Park Milton Keynes, England, United Kingdom, MK14 6FG</p>

Revision Record

Rev	Issued	Sections Affected	Detail
A	January 2024	All	New document.

