



Aperio GT 450 Release 1.3.3 Patch Notice

Leica Biosystems is always looking for ways to further improve our customers' experience using our products. Our software patches often include new features, but also include improvements based on customer input and our own internal testing. Please make sure to read below for the details of this patch so you can get the best out of your product. Thank you for being a customer of Leica Biosystems.

New Features Included in the GT 450 Patch

Aperio GT 450 release 1.3.3 now includes the following updates:

- Security patches and updates were applied to third-party off-the-shelf software.
- Compatibility support for the Microsoft Windows Server 2022 operating system is available.
- Compatibility support for newer SQL Server 2022 is available.
- Lab administrators can now configure the scanner to populate pyramid layers each 2x rather than the default 4x. This can improve users' viewing experience in some third-party viewers.

Improvements Included in the 1.3.3 Patch

Case #	Description
27884	Improves an image transfer NTP/scanner time sync issue.
28018	Replaces Mirth Connect with an alternative messaging interface. This resolves an issue that caused scanner halting issues for some customers.
28524	Removes a TIF file that is unexpectedly present in the DICOM Temp folder.
28527	Addresses a SAM issue where canceling an edit in Settings causes the system to hang and not complete with a verification message.
28607	Addresses an issue where a dark label appears on the scanner console.
28628	Changes index axis commutation filter to speed convergence.
28509	Restores Russian language support to the Aperio GT 450 console.
28293	Addresses a barcode decoding issue that caused a slide to be added to the incorrect specimen/patient.
28641	Addresses a barcode reader problem where characters are dropped.
28625	Addresses a barcode reader problem where characters are dropped.

Aperio GT 450 Release 1.3.3 Patch Notice
Part Number/Revision: PN-0065, Revision A | Date: July 2024
This document applies to GT 450 (RUO) release 1.3.3 and later.


Copyright Notice

- ▶ Copyright © 2024 Leica Biosystems Imaging, Inc. All Rights Reserved. LEICA and the Leica logo are registered trademarks of Leica Microsystems IR GmbH. Aperio, GT, and GT 450 are trademarks of Leica Biosystems Imaging in the USA and optionally in other countries. Other logos, product and/or company names might be trademarks of their respective owners.
- ▶ This product is protected by registered patents. For a list of patents, contact Leica Biosystems.

User Resources

- ▶ For the latest information on Leica Biosystems Aperio products and services, please visit www.LeicaBiosystems.com/Aperio.

Contact Information – Leica Biosystems Imaging, Inc.

Headquarters	Customer Support
 Leica Biosystems Imaging, Inc. 1360 Park Center Drive Vista, CA 92081 USA Tel: +1 (866)-478-4111 (toll free) Direct International Tel: +1 (760) 539-1100	Contact your local support representative with any query and service request. www.leicabiosystems.com/contact-us/

Revision Record

Rev	Issued	Sections Affected	Detail
A	July 2024	All	New document. Addresses CR 2024-0010.



23GT450
23GT450-R
23GT450SAMSW



00815477020228,
00815477020495
00815477020471